

## How to clean the pad

SKU : 386215G



This pad is called PN#386215G

This strap is called PN#609032G



Once the pad is zipped into the wash bag, make sure to use the gentle or delicate cycle on the washer in cool water.

# How to clean the Strap

---

SKU : 609032G

- It's machine washable. Wash it without detergent, only with cool water.
- Make sure to use the gentle or delicate cycle on the washer.
- Do not put the strap into the dryer after cleaning it.
- It's strongly suggested to hand wash the strap to maintain the strap quality.
- Fasten the hook-and-loop sides together and put it into the wash bag before placing it in the washer.  
(see the photos below)

STEP1



STEP2



STEP3

